

HELP SAVE A LIFE WITH...

NEW BEATY CPR FEEDBACK DEVICE

PART NO. 879411

Beaty was developed to provide real time CPR feedback to help bystanders and people to perform CPR in case they do not have any prior medical knowledge.



Beeps when compressions are effective



Great addition to any first aid kit or AED carry case



Smart and easy to use with no buttons or displays



Conveniently pocket-sized for quick access

SPECIFICATIONS

- Beaty runs on a 3v lithium battery that can last for more than 5 years on standby mode or 15 hours of continuous CPR
- Beaty should be used on patients 8 years and older

CONTACT US TO ORDER TODAY!

Accidental. First Aid, Health & Safety Solutions.

• Call: 1300 362 327 • Fax: 1300 882 122 • Web: accidental.com.au

1.



In case of CPR call for HELP.

2.



Place Beaty in the middle of patient's chest and Beaty will make a sound for adequate chest compressions performed, reaching a depth of 5cm.

3.



Push hard until a "beep" sound is heard. If no sound is heard, this means you need to push harder.

